


MERISCREEN Malaria Pf/Pan Ag

Evaluated
& Pre Qualified
by WHO

Meril

Diagnostics

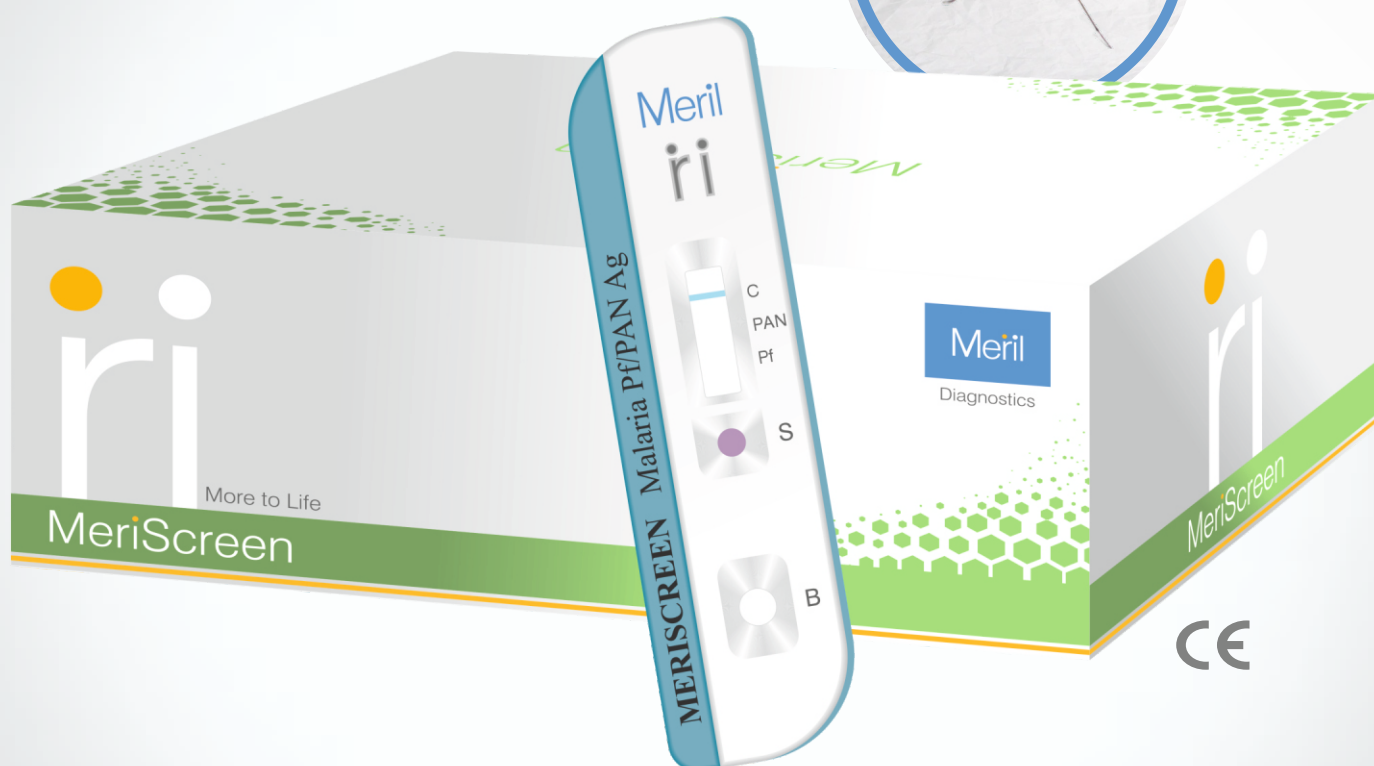

SPECIFICATION


Type	Rapid immunochromatographic test kit
Detected Species	P.falciparum and P. falciparum, P. vivax, P. malariae, P. ovale (through pan-pLDH antigens)
Specimen	Whole Blood
Specimen transfer device	Inverted Cup
Read Time	20 minutes
Storage Temperature	1 - 40°C
Shelf Life	24 months
Limit of Detection (LOD)	P. falciparum ≥ 50 parasites/ μ l and for P. vivax ≥ 200 parasites/ μ l

MERISCREEN Malaria Pf/Pan Ag




Diagnostics

TEST PROTOCOL

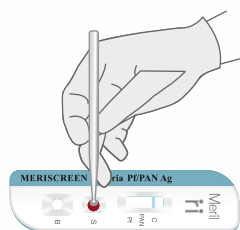


Add 5 µl of whole blood using Specimen transfer device

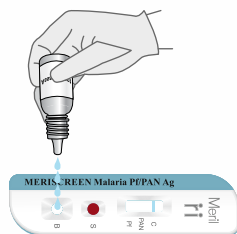

Dispense 4 Drops of Assay buffer

POCT

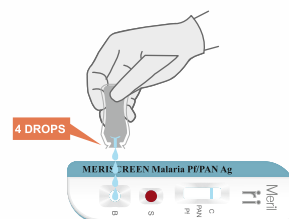

Dispense 4 Drops of Assay Buffer



At 20 minutes

INTERPRETATION



Positive for *P. falciparum* or a mixed infection with *P. vivax*, *P. ovale* and/or *P. malariae*



Positive for *P. falciparum*




Positive for Pan (*P. falciparum*, *p. vivax*, *P. ovale* or *P. malariae*)




Negative



Invalid. Take a new cassette packaging and repeat the test

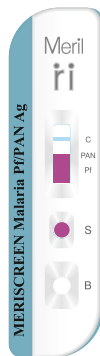

Invalid. Take a new cassette packaging and repeat the test

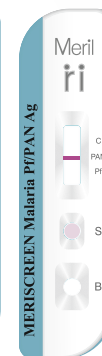

ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	Pack Size
MHLRPD - 02	MERISCREEN Malaria Pf/Pan Ag	30 Tests
MHLRPD - 03	MERISCREEN Malaria Pf/Pan Ag	10 Tests
MHLRPD - 04	MERISCREEN Malaria Pf/Pan Ag	25 Tests
MHLRPD - 05	MERISCREEN Malaria Pf/Pan Ag	50 Tests
MHLRPD - 06	MERISCREEN Malaria Pf/Pan Ag (POCT)	1 Test/kit
MHLRPD - 07	MERISCREEN Malaria Pf/Pan Ag (POCT)	1 Test/kit x 10 each
MHLRPD - 08	MERISCREEN Malaria Pf/Pan Ag (POCT)	1 Test/kit x 25 each



Meril Diagnostics Pvt. Ltd.

D1- D3. Second Floor. Meril Park. Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

+91 9624025718 | Email: mahendra.s@merillife.com | askinfo@merillife.com

WBRO/IMG/MHLRPD02/00
DATE:03/09/2024