



MERISCREEN Malaria Pf/Pv Ag

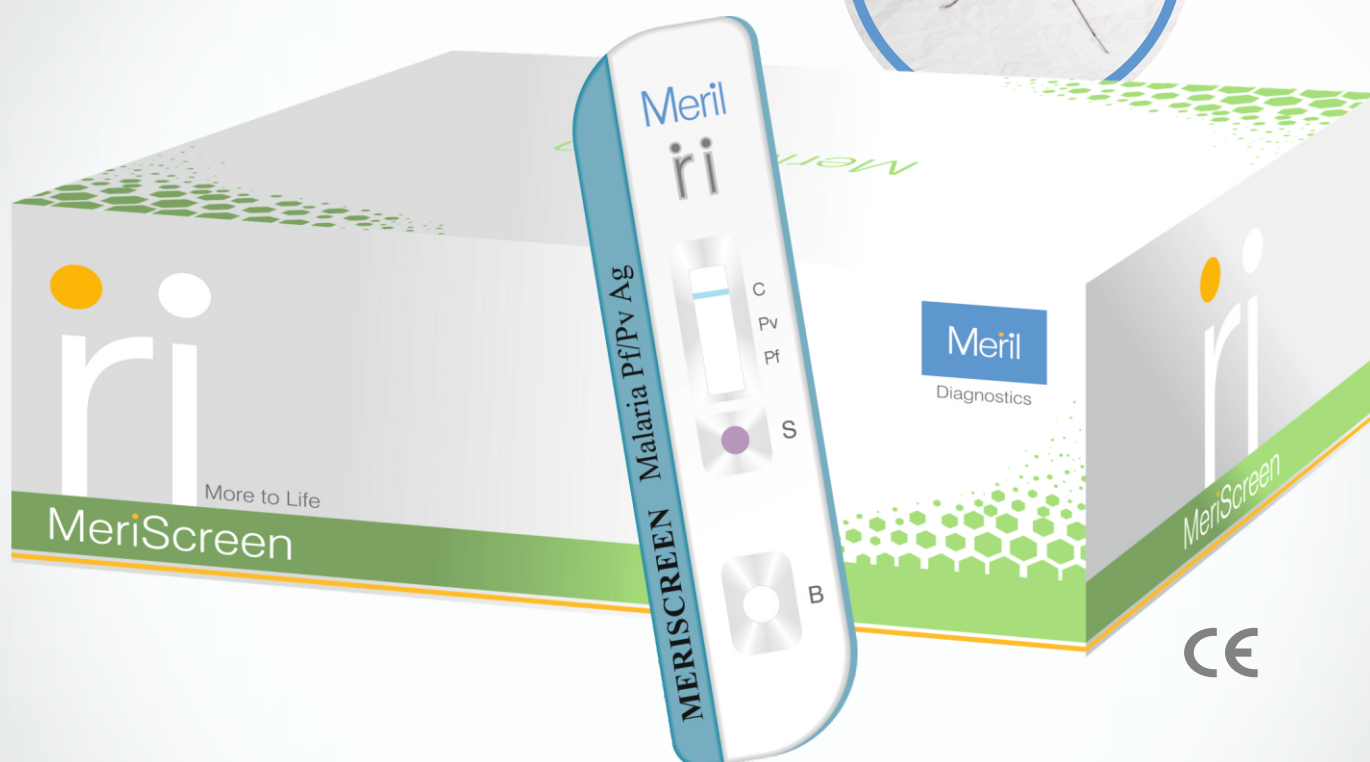


SPECIFICATION

Type	Rapid immunochromatographic test kit
Detected Species	P.falciparum and P.vivax
Specimen	Whole Blood
Specimen transfer device	Inverted Cup
Read Time	20 minutes
Storage Temperature	1 - 40°C
Shelf Life	24 months
Limit of Detection (LOD)	P. falciparum ≥ 50 parasites/ μ l and for P. vivax ≥ 200 parasites/ μ l

TEST PROTOCOL

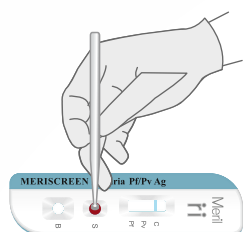


Add 5 µl of whole blood using Specimen transfer device

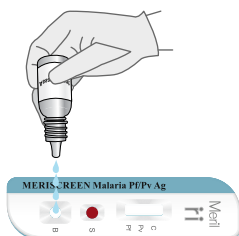

Dispense 4 Drops of Assay buffer

POCT

4 DROPS

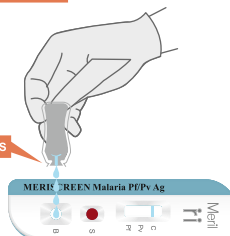

Dispense 4 Drops of Assay Buffer



At 20 minutes


INTERPRETATION



Positive for *P. falciparum*




Positive for as a mixed infection with *P. falciparum* and *P. vivax*

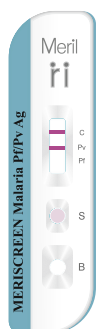


Positive for *P. vivax*




Negative

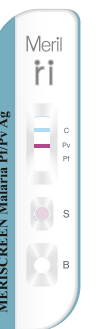

Invalid. Take a new cassette packaging and repeat the test

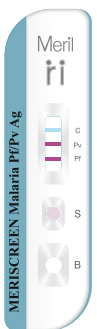

Invalid. Take a new cassette packaging and repeat the test

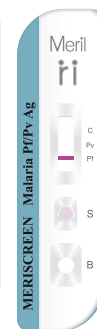

ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	PACK SIZE
MFLRPD - 02	MERISCREEN Malaria Pf/Pv Ag	30 Tests
MFLRPD - 03	MERISCREEN Malaria Pf/Pv Ag	25 Tests
MELRPD - 04	MERISCREEN Malaria Pf/Pv Ag	50 Tests
MFLRPD - 05	MERISCREEN Malaria Pf/Pv Ag	10 Tests
MFLRPD - 06	MERISCREEN Malaria Pf/Pv Ag (POCT)	1 Test/kit
MFLRPD - 07	MERISCREEN Malaria Pf/Pv Ag (POCT)	1 Test/kit x 10 each
MFLRPD - 08	MERISCREEN Malaria Pf/Pv Ag (POCT)	1 Test/kit x 25 each



Meril Diagnostics Pvt. Ltd.

D1- D3, Second Floor, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

+91 9624025718 | Email: mahendra.s@merillife.com | askinfo@merillife.com