


MOZECTM NC
Rx PTCA Balloon Dilatation Catheter

Meril

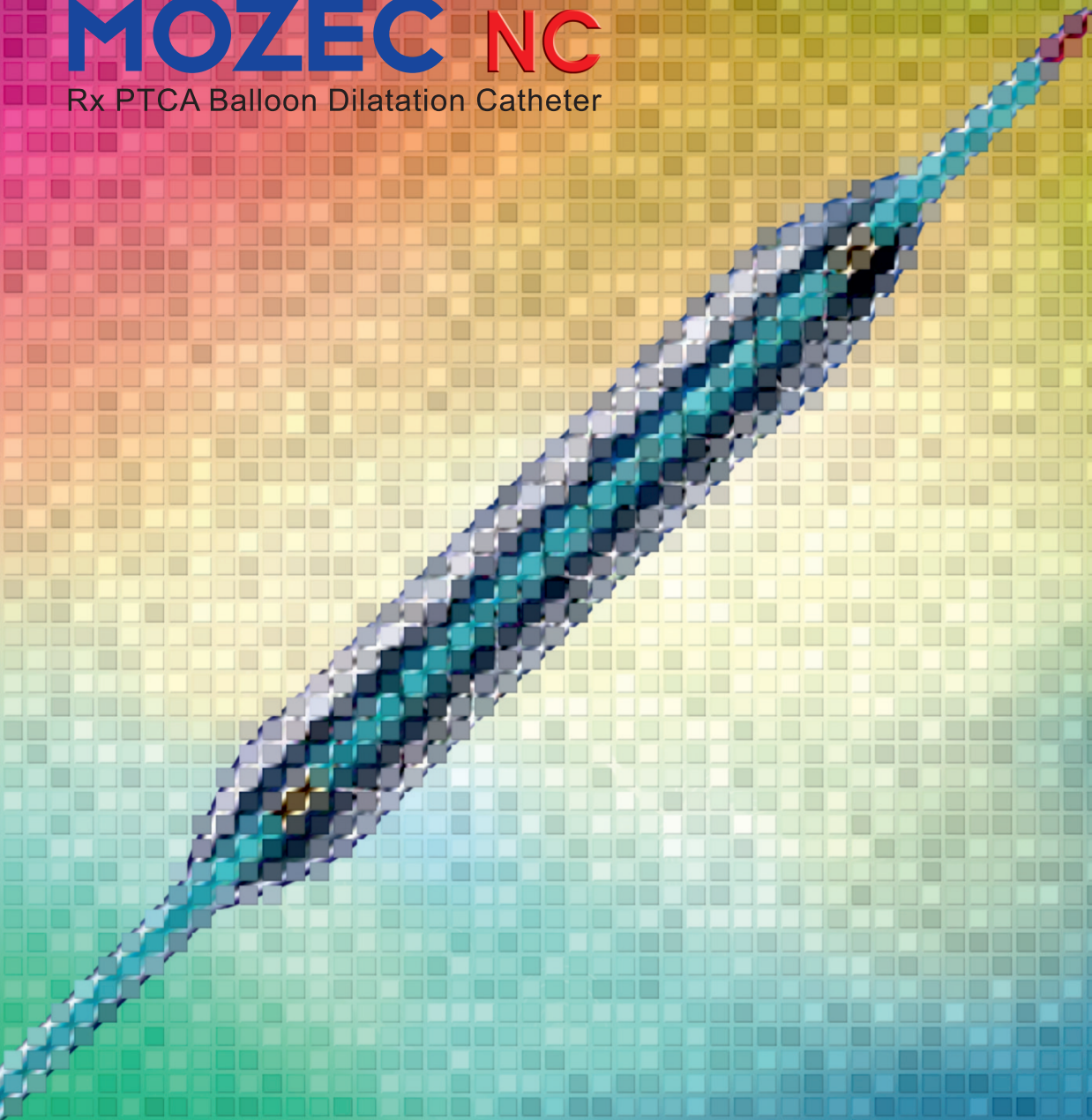

MOZECTM NC
Rx PTCA Balloon Dilatation Catheter

MOZECTM NC

Rx PTCA Balloon Dilatation Catheter

Handicraft for *Angiocraft*.

Experience controlled and confident expansion with MOZECTM NC



Soft tip made up of medical grade PEBAX to facilitate advancement of the catheter to and through the stenosis




Low tip entry profile for easy cross and re-cross




Hump free fusion of distal tip inner tubing and balloon



Biocompatible, hydrophilic, lubricious coating from distal balloon base upto rapid exchange port

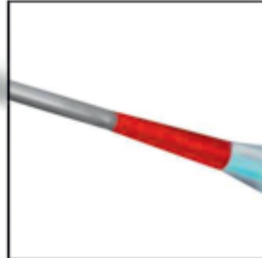

Short balloon shoulders
2 folds for Ø 2.00 mm
3 folds for Ø 2.25 to 5.00 mm

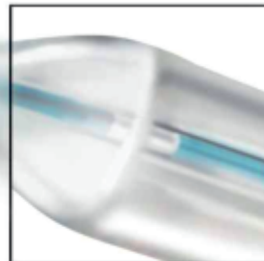

Nylon balloon material provides non-compliance at high pressure



Low crossing profile and highly flexible **MOZEC™ NC** is designed to acquire maximum pressure and meet clinical needs



Compliance Chart

Pressure (ATM)	2.00 mm	2.25 mm	2.50 mm	2.75 mm	3.00 mm	3.25 mm	3.50 mm	4.00 mm	4.50 mm	5.00 mm
6	1.84	2.18	2.24	2.50	2.76	3.03	3.38	3.65	4.03	4.48
8	1.91	2.20	2.35	2.60	2.82	3.11	3.42	3.82	4.22	4.68
10	1.96	2.23	2.42	2.68	2.91	3.18	3.46	3.94	4.39	4.86
12	2.00	2.25	2.50	2.75	3.00	3.25	3.50	4.00	4.50	5.00
14	2.03	2.28	2.54	2.77	3.03	3.29	3.54	4.08	4.58	5.10
16	2.06	2.30	2.56	2.79	3.06	3.33	3.58	4.14	4.66	5.18
18	2.09	2.32	2.58	2.83	3.10	3.37	3.62	4.18	4.74	5.24
20	2.11	2.35	2.61	2.86	3.13	3.40	3.66	4.23	4.80	5.29



NP



RBP

MOZEC™ NC BALLOON TECHNICAL SPECIFICATIONS

Catheter System	: Rapid Exchange (Rx)
Balloon Material	: Nylon (Non-Compliant)
Available Balloon Lengths (mm)	: 8, 10, 12, 13, 15, 18, 23, 28, 30, 35, 38
Available Balloon Diameters (mm)	: 2.00, 2.25, 2.50, 2.75, 3.00, 3.25, 3.50, 4.00, 4.50, 5.00
Usable Catheter Length	: 142 cm
Max Guidewire Compatibility	: 0.014" (0.36 mm)
Guide Catheter Compatibility	: 5F (min I.D. 0.056" / 1.42 mm)
Distal Shaft Coating	: Biocompatible, Hydrophilic & Lubricious coating from distal balloon base upto rapid exchange port
Balloon RO Markers	: Platinum / Iridium
Tip Length	: 3.5±0.5 mm
Proximal Shaft Diameter	: 1.98 F
Distal Shaft Diameter	: 2.7 F
Markers on Proximal Shaft	: Brachial and Femoral markers 90 cm and 100 cm from distal tip
Nominal Pressure (NP)	: 12 atm for all Diameters
Rated Burst Pressure (RBP)	: 20 atm for all Diameters

MOZEC™ NC ORDERING INFORMATION

Diameter\ Length	8 mm	10 mm	12 mm	13 mm	15 mm	18 mm
2.00 mm	MNC20008	MNC20010	MNC20012	MNC20013	MNC20015	MNC20018
2.25 mm	MNC22508	MNC22510	MNC22512	MNC22513	MNC22515	MNC22518
2.50 mm	MNC25008	MNC25010	MNC25012	MNC25013	MNC25015	MNC25018
2.75 mm	MNC27508	MNC27510	MNC27512	MNC27513	MNC27515	MNC27518
3.00 mm	MNC30008	MNC30010	MNC30012	MNC30013	MNC30015	MNC30018
3.25 mm	MNC32508	MNC32510	MNC32512	MNC32513	MNC32515	MNC32518
3.50 mm	MNC35008	MNC35010	MNC35012	MNC35013	MNC35015	MNC35018
4.00 mm	MNC40008	MNC40010	MNC40012	MNC40013	MNC40015	MNC40018
4.50 mm	MNC45008	MNC45010	MNC45012	MNC45013	MNC45015	MNC45018
5.00 mm	MNC50008	MNC50010	MNC50012	MNC50013	MNC50015	MNC50018

Diameter\ Length	23 mm	28 mm	30 mm	35 mm	38 mm
2.00mm	MNC20023	MNC20028	MNC20030	MNC20035	MNC20038
2.25mm	MNC22523	MNC22528	MNC22530	MNC22535	MNC22538
2.50mm	MNC25023	MNC25028	MNC25030	MNC25035	MNC25038
2.75mm	MNC27523	MNC27528	MNC27530	MNC27535	MNC27538
3.00mm	MNC30023	MNC30028	MNC30030	MNC30035	MNC30038
3.25mm	MNC32523	MNC32528	MNC32530	MNC32535	MNC32538
3.50mm	MNC35023	MNC35028	MNC35030	MNC35035	MNC35038
4.00mm	MNC40023	MNC40028	MNC40030	MNC40035	MNC40038
4.50mm	MNC45023	MNC45028	MNC45030	MNC45035	MNC45038
5.00mm	MNC50023	MNC50028	MNC50030	-	-

Disclaimer: Mozec™ NC Diameter 3.25mm, 5.00mm and length 12mm is CE approved.
Mozec™ NC is US 510(k) cleared (Refer IFU/Labels for approved sizes)

Mozec™ NC is registered trademark of Meril Life Sciences Pvt. Ltd.



More to Life

Manufacturer:

Meril Life Sciences Pvt. Ltd.
Survey No. 135/139, Bilakhia House,
Muktanand Marg, Chala, Vapi - 396 191.
Gujarat, India.
T +91 260 240 8000

Subsidiary companies:

Meril Life Sciences Pvt. Ltd.
301, A-Wing, Business Square,
Chakala, Andheri Kurla Road,
Andheri East, Mumbai 400 093
T +91 22 39350700
F +91 22 39350777

Meril, Inc.

2436 Emrick Boulevard,
Bethlehem, PA 18020
T +610 500 2080
F +610 317 1672

Meril South America

Doc Med LTDA
Al. dos Tupiniquins,
1079 - Cep: 04077-003 - Moema.
Sao Paulo, Brazil.
T +55 11 3624 5935
F +55 11 3624 5936

Meril GmbH.

Bornheimer Strasse 135-137,
D-53119 Bonn.
Germany.
T +49 228 7100 4000
F +49 228 7100 4001

Meril Tibbi Cihazlar

Meril Tibbi Cihazlar İmalat ve Ticaret A.Ş.
İçerenköy Mah. Çetinkaya Sok.
Prestij Plaza No:28
Kat:4 Ataşehir, 34752
İstanbul / Turkey
T +90 216 641 44 24
F +90 216 641 44 25

Meril China Co. Ltd.

2301b, 23f, Lixin Plaza,
no 90, South Hubin Road,
Xiamen, China
T 0086-592-5368505
F 0086-592-5368519

Meril SA Pty. Ltd.

102, 104, S101 and S102,
Boulevard West Office Park,
142 Western Service Road,
Erf 813 Woodmead Extension 17
Sandton, Johannesburg – 2191
South Africa
T +27 11 465-2049
F +27 86 471 7941

Meril Medical LLC.

Nauchnyi Proezd 19,
Moscow, Russia – 117 246.
Office - +7 495 772 7643

EU representative.

Obelis S.A.
Bd, General Wahis 53,
1030, Brussels, Belgium.
T +32 2 732 5954
F +32 2 732 6003
E mail@obelis.net

E askinfo@merillife.com
W www.merillife.com

