


ANCORA 360™

Heart Tissue Vacuum Stabilizer

Meril

Endo-Surgery

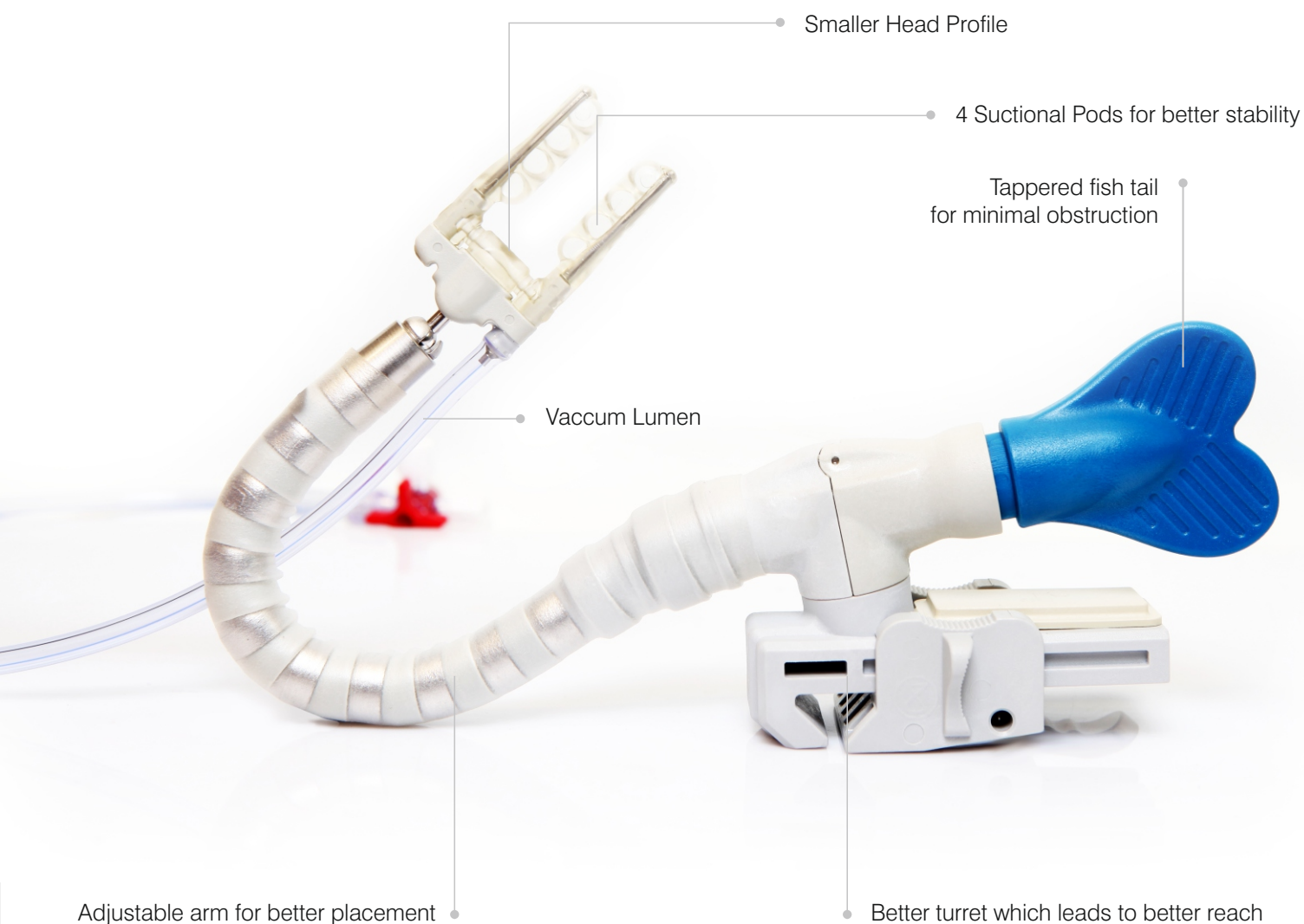

ANCORA 360™ tissue stabilizer includes features as clear, malleable pods and more flexible arm with reduced head profile.



Meril
Cardiac
Solutions

Meril Endo-Surgery Pvt. Ltd.

E1-E3, Meril Park, Survey No. 135/2/B & 174/2
Muktanand Marg, Chala, Vapi 396 191. Gujarat. India.
T: +91 260 3052 100 F: +91 260 3052 125
W: www.merillife.com



ANCORA 360™

Heart Tissue Vacuum Stabilizer

Meril

Endo-Surgery

Placement:

1. Attach the rail clamp to the surgical table. Position the rail clamp and mounting rail at the patient's hip.
2. Attach the mounting rail to the rail clamp.
3. Secure the flex arm in the mounting rail by locking the mounting rail lever.
4. Transfer the Suction and vacuum line extension to the sterile field using aseptic technique.
5. Connect the vacuum line extension to the luer connector of the canister tubing set and attach the canister tubing set to the regulated vacuum source. Set the regulated vacuum to (-)400 mm Hg.

