

Topical Skin Adhesive  
**MERIFIX**<sup>TM</sup>  
2-Octyl Cyanoacrylate

Meril

Endo-Surgery



**MERIFIX**<sup>TM</sup> Topical Skin Adhesive consists of monomeric 2-Octyl Cyanoacrylate, which polymerises quickly when comes in contact with tissue fluid.

**MERIFIX**<sup>TM</sup> Topical Skin Adhesive is intended for topical application to close easily approximated skin edges of wounds from surgical incisions, including punctures from minimally invasive surgery, and simple, thoroughly cleansed, trauma-induced lacerations.



**FIXING WOUNDS**  
WITH FINESSE

**MBS**<sup>TM</sup>  
Meril Biosurgical Solutions

## ADVANTAGES:

Excellent  
Strength  
& Flexibility

Easy  
Application

Faster  
Set time <  
130 seconds

Patient  
Comfort &  
Appreciable  
Cosmesis

Effective  
Microbial  
Barrier

## APPLICATION INSTRUCTIONS:

### Opening of MERIFIX™ pouch:

Pull apart pouch to expose the sterile, single patient use tube and tip. Hold tube with the tube opening pointing upward and lightly fix the tip on it. Once the tip is fixed tightly, the tube is ready for use.

### Preparation of wound:

Dry the moist wound prior to application, cleaning excessive blood and debris. Approximate the tissue edges and apposition during application.

### Process:

Lightly hold the wound together for about 30-40 seconds to allow MeriFix to settle. Subcuticular sutures may be used prior to application of MeriFix for securing wounds under tension

### Application of MERIFIX™:

Apply light pressure to the tube to expel MeriFix from the sterile tip. MeriFix is to be applied as a very thin film along the edges of the approximated wound, and not directly inside the wound.

## ORDERING INFORMATION:

Product Code	Product Content	Ordering Qty.
OCA-0.5	MeriFix -0.5 ml	
OCA-01	MeriFix -1 ml	6 tubes/ box

## USAGE INDICATIONS:

### Skin Closure for small to medium incisions (Upto 6 inches) in surgeries of

- ▶ General surgery – laparoscopy, hernia, appendicitis, vascular surgery
- ▶ Plastic surgery – cleft lip, excision of scars, skin graft
- ▶ Cardiovascular Surgery- artery/vein harvesting sites
- ▶ Gynecology – caesarean section, hysterectomy, tubectomy
- ▶ Pediatrics – management of pediatric lacerations



Meril

Endo-Surgery

MERIL ENDO-SURGERY PVT. LTD.

Corporate Headquarters, Vapi

E1-E3, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg,  
Chala, Vapi- 396 191, Gujarat, India.

Tel.: +91 260 3052 100, Toll Free No.: 1800-4194-433

India Sales and Marketing, Mumbai

601-612, Midas, Sahar Plaza Complex, M.V. Road, J.B Nagar,  
Andheri (East), Mumbai 400 059, India. Tel.: +91 22 307 32 400  
Email: endosurgery@merillife.com, Web: www.merillife.com